



# PRODUCT OVERVIEW

## 2024

LEG POSITIONER (FOOT SUPPORT) FOR KNEE SURGERY

SWISS MADE  SINCE 2000



**THE “MUST” FOR SURGERY OF THE KNEE JOINT**

Wirz Medical (Wirz Ortho Mechanics GmbH) | Bernstrasse 1 | CH-3076 Worb  
T +41 31 832 43 53 | M +41 79 630 66 44 | F +41 31 832 43 54 | [mechanik@wirz-ortho.ch](mailto:mechanik@wirz-ortho.ch) | [legholderrx.ch](http://legholderrx.ch)

# ADVANTAGES

- ✓ When used in combination with lateral post, it replaces an assistant.
- ✓ Six different, stable positions.
- ✓ Easy to use, no need to assemble and disassemble.
- ✓ No need to sterilize.
- ✓ No need for additional disposables.
- ✓ X-Ray transparent.




***This is the best invention in orthopaedics in the last 20 years!***



**Dr.med. Alois Lustenberger, Orthopaedic Surgeon, Orthomed Group Practice, Switzerland**

# PRODUCTS 2024

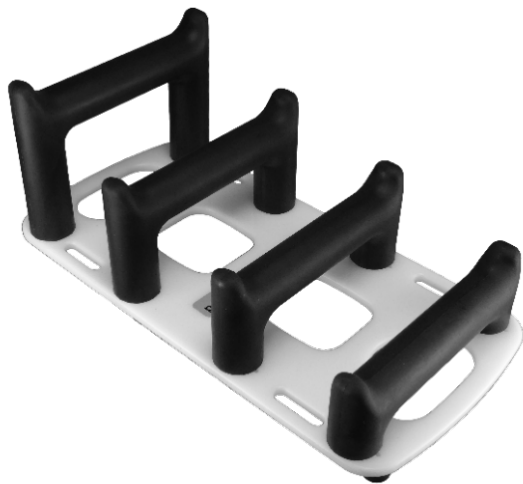


## LegholderRX® standard

The best-selling version of our leg positioners in the past, the LegholderRX® standard now features a more stable construction, optimized padding with side knobs and Velcro straps with marked pull tabs.

Article no.: LRX2020-02

GTIN Article: 7649994277024

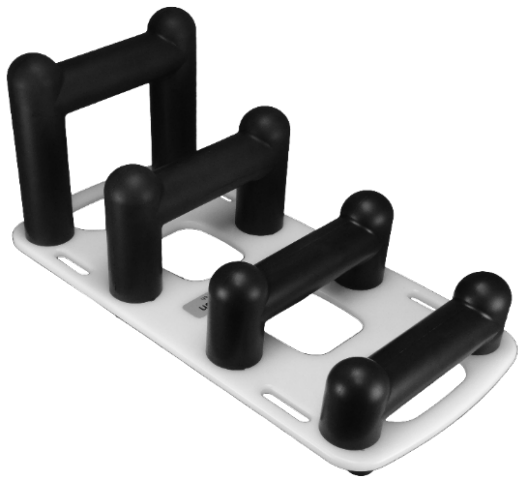


## LegholderRX® fusion


The new improved “fusion” edition of our LegholderRX® standard features special hemispherical knobs on the side of the stirrups to allow easier sliding of the drapes between the stirrups and to prevent the foot from slipping out sideways. The increased height between stirrups ensures enhanced stability.

Article no.: LRX2024-03

GTIN Article: 7649994277055

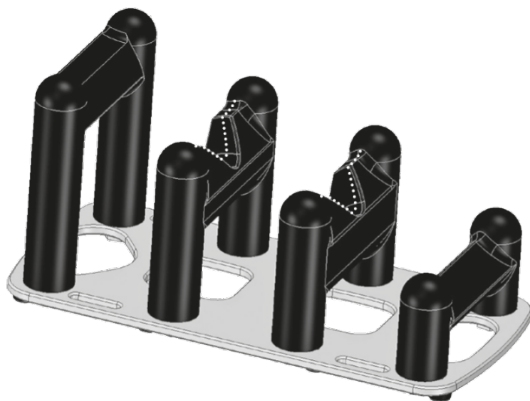

## LegholderRX® extra stable

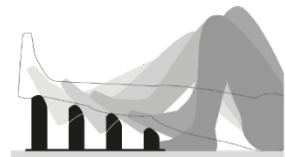

Very stable positioning thanks to biometrically calculated and ergonomic flat contact surfaces, increased height and recess for the Achilles tendon on the 2<sup>nd</sup> and 3<sup>rd</sup> brackets (dotted line).

Article no.: LRX2024-04

GTIN Article: 7649994277048



The increased overall height can be compensated by lowering the leg section.

\*Each product is delivered with long or short Velcro straps.

# TYPES OF SURGERIES THAT CAN BE PERFORMED WITH LEGHOLDERRX®



The LegholderRX® provides stable positioning of the foot and the leg, allowing surgery in **six different knee - joint flexion positions** without support in the poplitea. Fixation is performed directly by Velcro straps around the OR table before prepping.

This way of positioning in combination with a **lateral thigh post** is very helpful in **open knee surgeries such as arthroplasties, osteotomies, cartilage reconstruction and patellar surgery.**

Motion at the ankle joint is free. The malleoli and the center of the ankle joint are easily palpable, thus facilitating axial alignment.

“

*The models cannot be broken ... I personally really like the extra stable version. It offers additional stability, especially for bigger patients.*

”

Prof.Dr.med.Niklaus Friederich, University Hospital Basel





# NEW IMPROVED VELCRO STRAPS

The Velcro straps have a practical locking bar and visual directions on the pull tab to depict correct use.



# AVAILABLE IN TWO LENGTHS

## SHORT VELCRO STRAPS

Ideally used to fasten the LegholderRX® only around the one leg section on the operation side. This enables free movement of the leg section.



# LONG VELCRO STRAPS

Can be used over one leg section as well as over both leg sections.



## TECHNICAL DATA LEGHOLDER



**Dimensions**

490 x 240 x 200mm

**Weight**


3 kg, including two robust  
(long or short) Velcro  
straps

**Specifications**

Easy maintenance, robust,  
non-allergic plastic

**Cleaning**

No need to sterilize, inert  
against common cleaning  
agents

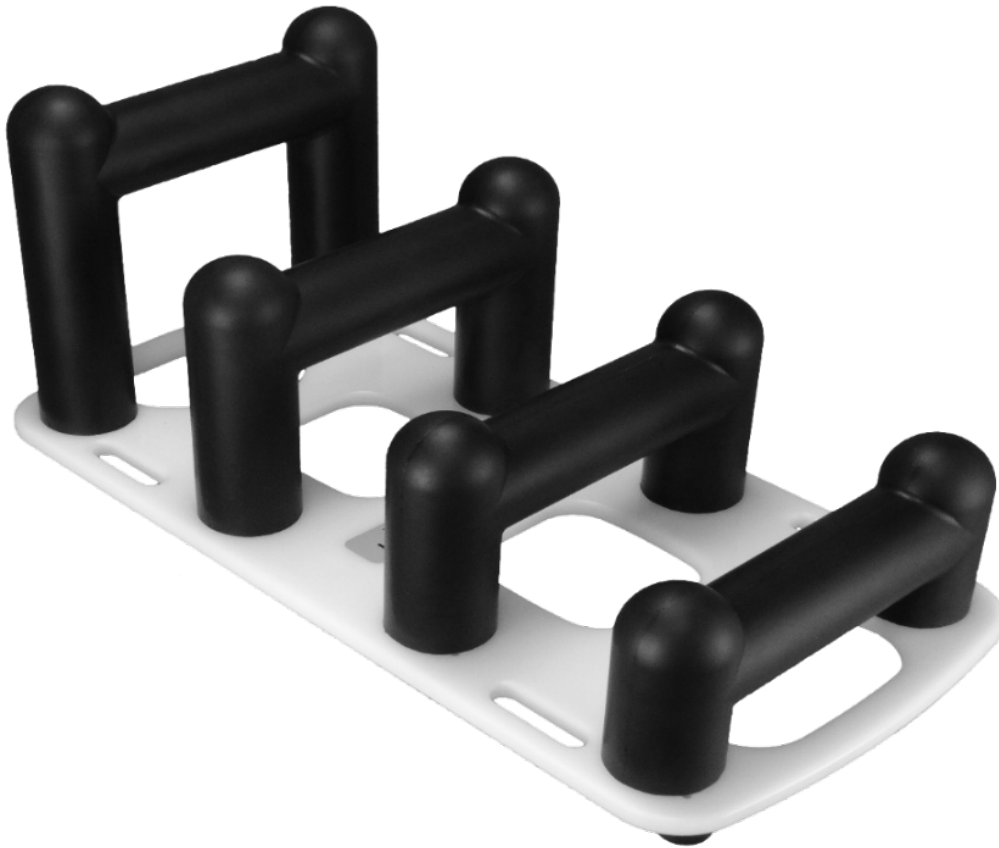

SWISS MADE  SINCE 2000

**Wirz Medical (Wirz Ortho Mechanics GmbH) | Bernstrasse 1 | CH-3076 Worb**  
**T +41 31 832 43 53 | M +41 79 630 66 44 | F +41 31 832 43 54**  
**[mechanik@wirz-ortho.ch](mailto:mechanik@wirz-ortho.ch) | [legholderrx.ch](http://legholderrx.ch)**

Swissmedic N°: CH201505-0035 | GLN: 7649 9942 77000 | UID (MWST/EORI): CHE-480.499.766  
Compliant to MDR 13/485 | Eudamed N°: APP000064674 | EU Representative: endocon.de | CE, ®, mod. dép.